

Colar Cervical

CO-100IS, CO-050S, CO-050M, CO-050L, CO-050XL

ANTES DA SUA COLOCAÇÃO LEIA AS SEGUINTESS CONSIDERAÇÕES:

- A duração do tratamento deve ser decidida exclusivamente por um médico.
- Respeite expressamente a duração diária de utilização deste produto seguindo as indicações médicas.
- Não utilize medicamentos de uso tópico na zona de colocação deste produto sem autorização médica.
- Utilize o produto com a medida adequada.

PRODUTO LAVÁVEL:

- Proteger os velcros fechando os mesmos antes da lavagem.
- Pode ser lavado à mão ou à máquina. Usar água morna.
- Não usar lixívia. Não engomar. Não lavar.
- Secar sobre uma superfície horizontal.

REF: CO-100IS - Comp/Alt: 32x4cm - Uni./cx: 16

REF: CO-050S - Comp/Alt: 37x8,5cm - Uni./cx: 25

REF: CO-050M - Comp/Alt: 40x8cm - Uni./cx: 55

REF: CO-050L - Comp/Alt: 46x8,5cm - Uni./cx: 45

REF: CO-050XL - Comp/Alt: 51x8,5cm - Uni./cx: 40

REF: CO-100IS - Colar Cervical semi rígido neonatal - CDM: 47240148

REF: CO-050S - Colar Cervical Tam. S - CDM: 60230304

REF: CO-050M - Colar Cervical Tam.M - CDM: 31954219

REF: CO-050L - Colar Cervical Tam.L - CDM: 42612489

REF: CO-050XL - Colar Cervical Tam.XL - CDM: 31954227

FABRICANTE:

2miil
MEDICAL DEVICES

2miil artesanía sanitaria, s.l.
C/ Talleres n31, Pol. Ind. 28430, Alpedrete, Madrid
www.2miil.com | 2miil@2miil.com
Tlf: (+34) 91 630 92 08 | Fax: (+34) 91 279 36 59

